

BLEPHA CLEAN® PF

**Sterile and 100% plant-based
wipes*, soaked with micellar
cleansing lotion.
For daily eyelid hygiene and
sensitive skin.
Very good tolerance.**

PACK CONTAINS

20 pre-impregnated sterile
plant-based wipes* for single
use only.

For cutaneous application.

* *Cellulose fibres*

PROPERTIES AND INDICATIONS

BLEPHACLEAN® PF is
recommended for the daily
hygiene of eyelids and sensitive
skin and to keep eyelids clean
before and after ophthalmic
procedures.

BLEPHACLEAN® PF gently
cleanses and removes dust,
dirt, pollen and make-up,
without irritating the eyes or
damaging the outer layer of the
skin.

BLEPHACLEAN® PF has a mix of
ingredients to **soothe sensitive
skin**.

BLEPHACLEAN® PF is for
the daily hygiene of eyelids
especially:

- when you have sensitive skin,
eyelids or eyes.
- to respect the skin around the
eyes and its natural bacterial
flora.

Tolerance has been tested
under ophthalmological,
dermatological and paediatric
control.

Fragrance-free and non-greasy,
BLEPHACLEAN® PF is ready to
use and does not need to be
rinsed after use.

BLEPHACLEAN® PF can be used
by contact lens wearers.
BLEPHACLEAN® PF can be used
by the entire family and on
babies over 3 months old.

Instructions for use:

Use twice a day, morning and
evening.

Use one wipe for each eye.

If you wear contact lenses, you
must remove them before
using BLEPHACLEAN® PF.

Wash hands thoroughly before
using the wipes.

Tear across the sachet to open
and unfold the wipe inside
(Figure 1); do not use any tool
to open the sachet.




Figure 1: Opening the sachet.

Wrap the wipe around the top
of your index finger.
Keeping your eye closed,
carefully place the wipe on your
eyelids and the base of your
eyelashes (Figure 2).
Gently massage with small
circular movements, removing
any crusts, secretions or make-
up.



**Figure 2: Cleansing your
eyelids and eyelashes.**

In front of a mirror, gently pull
your lower eyelid down with
your other hand and then
rub your finger with the wipe
wrapped around it along edge
of your lower eyelid (Figure 3).

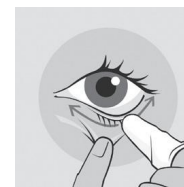

**Figure 3: Cleansing the eyelid
margin.**

**Repeat on the other eye,
using a new wipe.**

There is no need to rinse after
using BLEPHACLEAN® PF wipe.

Precautions for use:

For cutaneous application.

Do not apply directly into the
eyes.

Keep out of the sight and reach
of children.

The colour of the wipe may
vary. This is due to natural plant
extracts and is normal.

Ingredients:

AQUA, PEG-8, POLOXAMER
184, POLYSORBATE 20,
CAPRYLOYL GLYCINE,
PROPYLENE GLYCOL,
PEG-6 CAPRYLIC/CAPRIC
GLYCERIDES, CENTELLA
ASIATICA EXTRACT, IRIS
FLORENTINA ROOT EXTRACT,
SODIUM HYALURONATE,
DIPOTASSIUM PHOSPHATE,
POTASSIUM PHOSPHATE,
ALCOHOL, ZINC SULFATE,
SODIUM HYDROXIDE,
HYDROCHLORIC ACID.

Manufacturer:

Laboratoires Théa

12, rue Louis Blériot

63100 Clermont-Ferrand

France

Tel. +33 (0)4 73 98 14 36

UK RP and marketed in UK

by:

Théa Pharmaceuticals Ltd

MED IC5 Innovation Way

Keele University Science &

Innovation Park

Keele - Staffordshire - ST5 5NT

UK

[www.thea-pharmaceuticals.](http://www.thea-pharmaceuticals.co.uk)

[co.uk](http://www.thea-pharmaceuticals.co.uk)

0345 521 1290

Made in France

©2023, Laboratoires Théa.

All rights reserved.

Théa®, BLEPHACLEAN® are
registered trademarks of
Laboratoires Théa.